

# PRESSURE CARE

## QubiCell PM 265



### Active Surfaces

#### ***PM265 Alternating Mattress Overlay System***

The PM265 Alternating Overlay System is a high quality pressure redistribution surface suitable for clients with medium risk of Pressure Injury. Designed to fit onto a single bed (mattress overlay) this product is easy to install with basic instructions printed on the pump unit. Effective pressure redistribution is achieved by the alternation of the main body cells in a cycle of 9.6 minutes.

The elegant pump unit has a soft touch panel that is modern and simple to use. Features of the pump unit include 10 comfort level adjustments, incremented in weight levels for a range of client's needs. Static mode with auto revert after 20 minutes, low pressure alarm indicated by an audible sound and visual LED indicator. Smart pressure detection technology starts and stops the pump as required, therefore increasing the pump life expectancy, and reducing noise. Multi therapy modes assist carers and therapists in providing appropriate pressure care and improving client compliance. The lock out feature ensures peace of mind once unit is correctly set to individual needs. A power failure alarm and service alarm alert users or carers of the need for urgent action.

The mattress overlay incorporates 17 cells, with the top 3 cells providing a non alternating pillow section. The remaining 14 cells have a combination of ventilated and non ventilated cells. The ventilated cells allow for low air loss circulation assisting in reducing perspiration and increasing client comfort. The fully enclosed zippered cover is stretchable, vapour permeable, waterproof, antimicrobial with welded seams consistent with Australian infection control guidelines.

The PM265 Alternating Overlay System has been designed to assist with client safety, including a clearly indicated head and foot end, CPR release valves and alarms.

***The PM265 Alternating Overlay System assists in both prevention and healing of stage 1 & 2 pressure injuries or for clients at low to medium risk.***



#### ***PM265 Alternating Overlay System***

Overlay size 200cm (L) x 90cm (W) x 12.7cm (H)

***Safe working load of 160kg***

Registered on the Australian Register of Therapeutic Goods No. 133718

